ACCU-CHEK®
SmartGuide

**MOBILE DEVICE
AND SMARTWATCH
COMPATIBILITY**

**Contents / Inhaltsverzeichnis / Table des matières / Indice / Índice de contenidos /
Índice / Inhoudsopgave / Indhold / Spis treści / Obsah / Kazalo vsebine / Sadržaj /
Cuprins / İçindekiler**

ENGLISH

DEUTSCH

FRANÇAIS

ITALIANO

ESPAÑOL

PORTUGUÊS

NEDERLANDS

DANSK

POLSKI

ČEŠTINA

SLOVENŠČINA

HRVATSKI

ROMÂNĂ

TÜRKÇE

Mobile Device Compatibility

The Accu-Chek SmartGuide app and the Accu-Chek SmartGuide Predict app are compatible with most mobile devices that support the following system requirements. Check these system requirements before you install and operate the app on your mobile device.

- Your mobile device must support Bluetooth Low Energy 5.0 or higher.
- Your mobile device must run one of the following operating system (OS) versions (beta OS versions are not supported):
 - For iPhones, the iOS version must be iOS 15.3 or higher.
 - For Android devices, the OS version must be Android 11 or higher.

NOTE

There is a temporary pairing issue between a specific combination of Accu-Chek SmartGuide sensors and mobile devices.

- **The affected sensors** have a serial number (SN) higher than 1R000273961 and a manufacturing date before 2026-03-15. Sensors manufactured on or after 2026-03-15 are not affected. You can find the serial number and the date of manufacture  of your sensor on the bottom side of the Accu-Chek SmartGuide device packaging, or on the sensor applicator.
- **The affected mobile devices** are equipped with certain Exynos chipsets, which are especially found in certain Samsung Galaxy models. You can check the complete list of affected mobile devices here: <https://tools.accu-chek.com/documents/dms/smartguide-compatibility-devices/smartguide-compatibility-device-list.pdf>

Smartwatch Compatibility

If you use an iPhone, you can use the Accu-Chek SmartGuide app in combination with an Apple Watch.

For Apple Watches the watchOS version must be watchOS 9 or higher.

Other smartwatches are not compatible with the app.

Printed Version

If you would like a printed version of this document, contact customer support. The printed version is free of charge and will be sent to you within a few days.

Download

Download this document while connected to the Internet and save it to your mobile device for situations without an internet connection. This document is available to download from www.accu-chek.com.*

* download may incur data usage/charges

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ACCU-CHEK and ACCU-CHEK SMARTGUIDE are trademarks of Roche.

Apple Watch, watchOS, and iPhone are trademarks of Apple Inc., registered in the U.S. and other countries.

App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

IOS is a trademark or registered trademark of Cisco in the U.S. and other countries.

Android, Google Play and the Google Play logo are trademarks of Google LLC.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license.

All other product names and trademarks are the property of their respective owners.

© 2026 Roche Diabetes Care

Roche Diabetes Care GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany

www.accu-check.com

Last update: 2026-03
1000050884(04)